

Make 'n Take Sheet

Discover How Easy It Is To Make Your Own Assistive Technology!

PVC Pencil/Marker Grip



Materials:

2 – 1/2" PVC Pressure Tee Connectors	Add protective covering on thumb screw (ex: pencil grip)
1 – 1/2" PVC Nipple, or cut 4" from length	(PVC and other materials are available at Home Depot, Lowe's or similar hardware stores. Pencil grips can be found at Wal-Mart)
1 – 1/4" x 1" thumb screw	

Directions:

1. Drill a 1/4" hole into one of the PVC Tees. Locate the hole at the center of the tee (see diagram).
2. Assemble the pencil holder by attaching a PVC Tee to each end of the PVC Nipple. The parts may be threaded and will screw together.
3. Thread the thumbscrew into the drilled hole.
4. Place pencil, marker or paintbrush into the Tee and tighten with the thumbscrew until secure. Add protective covering over thumbscrew.

Oklahoma ABLE Tech, the statewide Assistive Technology Act Program, is funded through the Administration for Community Living of the U.S. Department of Health and Human Services, and maintains coordination and collaboration efforts with partners throughout the State of Oklahoma.

To contact Oklahoma ABLE Tech call 1-800-257-1705