


Make 'n Take Sheet

Discover How Easy It Is To Make Your Own Assistive Technology!

iPad Keyguard Overlay

A keyguard for an iPad can be made from a transparency sheet and Wikki Stix to decrease unwanted activation of buttons. The Wikki Stix can be placed where needed for any app. This make 'n take will utilize a symbol communication board and a sheet protector in the same way.

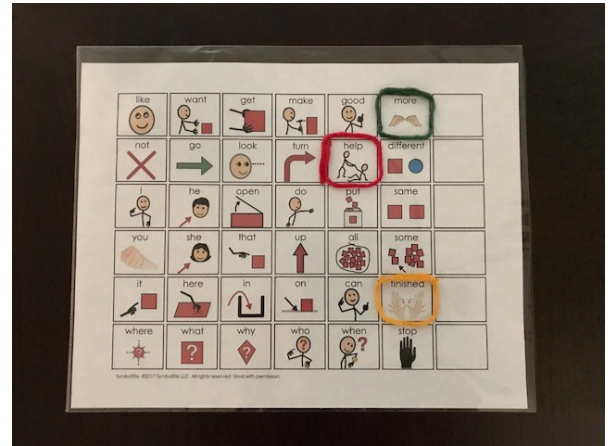

Materials:

- 1 – communication board
- 2-4 Wikki Stix of different colors
- 1 - sheet protector

Directions:

1. Slide communication board into sheet protector.
2. Identify 2-4 symbols the student typically needs to use or that you want to encourage use of.
3. Use Wikki Stix to outline the square of each symbol. Wikki Stix can be cut to desired length or you can use more than one Wikki Stix if the square is larger.

When making a keyguard for an iPad, place transparency sheet over iPad when app is in use. Identify areas you wish to isolate and place Wikki Stix accordingly. Depending on which iPad case is used, you may be able to place the transparency inside the case to secure it.

Oklahoma ABLE Tech, the statewide Assistive Technology Act Program, is funded through the Administration for Community Living of the U.S. Department of Health and Human Services, and maintains coordination and collaboration efforts with partners throughout the State of Oklahoma.

To contact Oklahoma ABLE Tech call 1-800-257-1705.